SL EarPro prilo
20 ml / bistra tekočina

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ENG EarPro * Spray *
20ml / clear liquid

Package Insert * Information for user
Please read entire package insert before use of EarPro Waterproofing Ear Spray. Keep this package insert. You may need to refer to it again. If any of the side effects become serious, or if you notice any side effects not listed in this leaflet, cease use and consult your Doctor. Please contact us on info@earpro.co

In this package insert:
1. What is EarPro and what it is used for?
2. Ingredients
3. Warnings: Before you use EarPro
4. Directions: How to use EarPro?
5. Possible side effects
6. How to store EarPro

1. WHAT IS EARPRO AND WHAT IT IS USED FOR?

• EarPro ear spray is used to help **prevent water from getting trapped into the ear canal**.
• EarPro is suitable for use by **children and adults of all ages**.
• Trapped water in the ear is a quite common problem, and can cause problems such as: discomfort, pain, inflammation and swimmers-ear. EarPro creates a **water repellent coating** and leaves the ear canal naturally.
• EarPro is widely used by **swimmers, families, divers, surfers and competitive athletes**.

2. Ingredients:

The ingredients in EarPro are: Solution based on paraffin oil and oregano oil.

3. WARNINGS: Before you use EarPro

Do not use EarPro:

- if you are **allergic (hypersensitive)** to paraffin oil or oregano oil
- if you have a **perforated ear drum or ear infection (e.g. after ear surgery, ventilation tubes)**
- if there is anything else wrong with your ear such as **pain, swelling, discharge**
- Avoid spraying in eyes, nose or mouth if EarPro gets into the eyes**
- Take care to avoid getting EarPro in your eyes.
- If this happens, rinse the eye immediately with water for 10 to 15 minutes, holding the eyelids apart and avoiding getting the rinse liquid in the other eye.
- Tell your doctor or pharmacist if irritation persists.
- Pregnancy and breast-feeding**
Ask your doctor or pharmacist for advice before using it. There is no experience or research on the use of EarPro in pregnant or lactating women. Therefore, the product should not be used in case of doubt.
- EarPro is for single person use only, don't share.**
- Do not spray near naked flame.**

4. Directions: How to use EarPro?

Use EarPro only in the ears. Warm in palm of hand, shake well. Do not point the nozzle at yourself or anyone else.
• Add 1 spray of EarPro to each ear
• Softly rub the tragus (outer skin) to make sure

the canal is coated.
• Then wipe away any surplus with a tissue.
• Wash hands after use.
• **Do not "plug" the ears** with loose cotton wool.
• **Re-apply after 2 hours** of water exposure
• Always return the bottle to the carton after usage.

5. POSSIBLE SIDE EFFECTS

No side effects are known if the product is used typically only according to the instructions for use. If wallowed side effects may occur and you should consult a doctor.
Stop using this product and tell your doctor or pharmacist if you notice any side effects not mentioned in this leaflet.

6. How to store EarPro?

- Keep out of the reach and sight of children.
- Do not use after the expiry date shown on the bottle and the carton. The expiry date refers to the last day of that month.
- Once opened, do not keep the bottle for longer than 6 months.
- Do not store above 45°C.
- Store the bottle upright in the outer carton to avoid spillage.

Questions or comments:
www.earpro.com

LiquidPro Ltd
166 Old Bakery Street
Valletta VLT1455 Malta
Rev: 05/2021 V1.1

EarPro is a registered trademark of LiquidPro Ltd.

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EarPro est une marque du groupe LiquidPro Ltd.