CE

www.medikoe.com

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Prepare your Me100 nebulizer system for use

- Ensure that the appliance and all accessories are completely clean.
- Place the appliance on a level and secure surface (e.g. a table top) with easy access to an electrical socket.
- Take the accessories out the plastic bag.
- Unwind the power supply cord by cutting the plastic tie and plug it in the socket located at the back side of the device.
- Take the tubing and attach one end to the device's air outlet and the other end to the bottom of the medication chamber.
- Unscrew the upper section of the medication chamber by turning it anti-clockwise.
- Put the medication to be nebulized into the chamber, measuring it according to the scale marked on the side of the chamber.
- Replace the upper section-cover of the chamber, securing it by turning it clockwise.
- Assemble the selected accessory; mask or mouthpiece on the upper section of the medication chamber.

Your device is now ready for use.

Aerosol therapy treatment

- Plug your device into a socket of the correct voltage.
- Press the switch of the device to position 1.
- Start the treatment and carry on for as long as you can still see the 'mist' produced by the inhalation of the medication performed by the appliance.
- Warning: The presence of a small amount (about 0,4 ml) of medication in the chamber at the end of each treatment is absolutely normal. This amount called residue cannot be nebulized.

End of each aerosol therapy treatment

- Turn off the device by pressing its switch to position 0.
- Unplug the appliance from the power socket.
- Take the power cord out of the unit.
- Detach the selected accessory from the medication chamber.
- Detach the flexible tubing from both the medication chamber and the device air outlet.
- Wipe the accessories and medication chamber clean according to the instructions written in instruction manuals for this unit.

Standard accessory kit

- Adult face mask
- Child face mask
- Mouth piece
- Flexible tubing
- Air filter
- Medication chamber – nebulizer

Cleaning and disinfection

Wash your hands well before beginning to clean and disinfect the accessories. Do not wash the device under the tap or soaking. If it needs cleaning just use a cloth damped with detergent (Not abrasive). Device with enclosure is not protected against the penetration of liquids. Changing the filter: lift up the filter lid. Remove the old filter and replace it with the new one.

Microbe contamination

If the presence of conditions carrying the risk of microbial infection and contamination, personal use of the accessories and nebulizer only is advised (always consult a doctor).

UK

Me100

Instruction manuals Me100 piston nebulizer

Please carefully read the instructions and warnings given in this handbook before use, and always keep it with the nebulizer.

Problems, causes and remedies

- The device does not turn on:
 - Ensure the plug is well into the socket
 - Ensure the switch is in the position (1).
- The device is alright but does not nebulize
 - Ensure that medicine chamber is completely assembled
 - Ensure that air connection tube is not crushed or bent.
 - Ensure that air suction filter is not blocked or dirty. If necessary, replace with a new one.
 - Ensure that required drug is inside medication chamber
- The device does not work
 - The thermal protector has triggered for the following reasons:
 - The device has worked outside the operating limits given in this manual.
 - The device has been used near to source of heat or in rooms with a temperature above 40°C. Leave it cool down for at least one hour and then turn it back on.
 - N.B. if the device still does not work correctly, contact the retailer.

Technical specification

- Piston electro compressor with the thermal cut – out
- Class I device
- Type B device
- Important – please check the operation instructions
- Alternating current
- Do not use the device while having a bath or shower
- Nominal technical data:
 - Voltage (230V 50Hz 150 VA)
 - Fuses (T 2,5A – 250V)
 - Working pressure: 1,0 – 1,3 bar
 - Maximum pressure 3,0 bar approx.
 - Operating flow: 6,5 l/min
 - Noise level: less than 57 dB/m
 - Neb rate: 0,50 ml/min
 - MMAD: 4,89µm
 - Dimensions: 210 x 145 x 110mm
 - Weight: 1,27 kg
 - Temporary use: 30/30min. ON/OFF
 - Professional suction filter: 50/70 micron
 - Class of risk: Ia
- Working conditions:

Warranty

The legal provisions in this respect apply. The warranty is limited to defective material and workmanship. Batteries and cable are not covered by the warranty. The warranty is only valid if the product has neither been opened nor subjected to violence or willful damage and it is returned with the original receipt. Contact your dealer if you have any complaints. For further information contact the customers service www.medikoe.com. In the event of warranty claims being demanded to be justified, the customer concerned will be supplied with a replacement product. MEDIKOEL cannot be held liable for any consequential damage.

Nebulizer instructions

Designed and created in accordance with the latest standards in an Medical Device (Risk class IIa – MDD 93/42/EEC) highly efficient in medical therapies administered by means of aerosol nebulization.

Safety warnings

- A single-patient use is recommended
- The device should be used only with compressor nebulizers for aerosol therapy that comply with standards in force and whose use is indicated in the instruction manual
- Device is not suitable for anesthesia and lung ventilation
- The device should be used only with original accessories shown in this manual
- Information concerning connection and use with device for aerosol therapy is shown in the instruction manual of the device. Always observe instructions and safety warnings as indicated in the instruction manuals of the device for aerosol therapy
- Use the device and its accessories as per your doctor's recommended
- Use medications prescribed by your doctor according to this indications
- Do not use the device for other use than nebulizer for aerosol therapy. Manufacturer will not be held responsible for inappropriate use of the device
- The device does not come in a sterile pack; always carry out cleaning and sterilization operations before and after each treatment. When using the device follow ordinary hygiene and cleaning precautions, (wash your hands and carry out cleaning operations as described in specific sections of this manual)
- The device has small components which might be removed and easily swallowed. Use by minors and disabled persons requires presence of an adult with his faculties. Do not leave the device unattended in places easily accessible by minors and disabled people.
- Always store the device in a dry and clean place far from light, heat and weather conditions.
- Dispose of device as provided by standards in force
- In presence of pathologies with microbial contamination and infection hazard a single patient use of accessories and nebulizer is recommended (always consult your doctor).

How to prepare and to operate the device

Please see chapter 'Prepare your Me100 nebulizer system for use above.'

Important: never inhale in horizontal position. Never bend the nebulizer over 60°.

Cleaning instructions

After each treatment clean thoroughly each component of the nebulizer removing medication residuals and possible impurities. Clean each component as below except the air tube which-in case of treatment to new patient or in case of impurities should be replaced.

Wash all components (except air tube) under tap warm water (about 60°C-140°F) for about five minutes adding if necessary a small quantity of detergent following dosage and use limitations as provided by detergent manufacturer.

Rinse thoroughly making sure that all deposits are washed away and let dry. All nebulizer components can be disinfected (except air tube) with chemical disinfection following dosage and use limitations as provided by disinfectant manufacturer.

They are usually available at pharmacy shops. All nebulizer components can be heat steam sterilized (except air tube and mask) to 121°C (20 min) or 134°C (7 min), by EN554/IS011134. The sterilization package must be confirmed by EN686/IS011607 and be suitable for steam sterilization. After sterilization always let all nebulizer components cool down to ambient temperature before use. Do not repeat sterilization cycle when nebulizer components are still warm.

Accessories life edge
It is advisable replace accessories after some 100 to 120 treatments on single patient or after about 20 sterilization cycles.

Storage and transportation conditions

- Storage and transportation temperature: min -25°C; max 70°C
- Storage and transportation humidity percentage: min 10%; max 95%

Operating technical data

- Working pressure: 1,0 – 1,3 bar
- Maximum pressure: 3,0 bar approx.
- Operating flow: 6,5/l min
- Noise level: less than 57 db/m
- Neb rate: 0,50 ml/min
- MMAD: 4,89μm
- Min. capacity: 1 ml
- Max capacity: 10 ml

⚠ Read carefully instructions for use.

Optional accessories

Nasal aspirator Me126 - a device for cleaning children nozzle

Nasal aspirator Me126 is the most advanced device in this field because it allows conventional aspiration through inhalation of air through the mouth, or the simplest known way of aspiration with help of piston nebulizer Me100. The user for aspiration purpose uses the suction power of the nebulizer unit with connecting of the nasal aspirator Me126 on the suction entrance of the nebulizer Me100.

Why is it advisable to use a nasal aspirator for babies?

Infants up to one year of age only breathe through nose and they are not able to blow out their own mucus.

Eliminated mucus in the baby nose makes difficulties at breathing. The nasal aspirators Me126 helps at removing the mucus out from infant nozzle. Clean and transitional nostrils allow an infant a peaceful sleep and easier breast-feeding.

Manufacturer: Medikoe d.o.o., Jajnova cesta 2, 4240 Radojvica, Slovenia, www.medikoe.com
Made in Slovenia CE0120 (Dir. 93/42/EEC)

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