naildoc® naildoc®

RAPID NAGELPILZ-STIFT

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


naildoc® RAPID

091648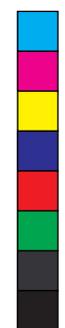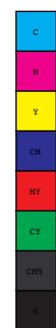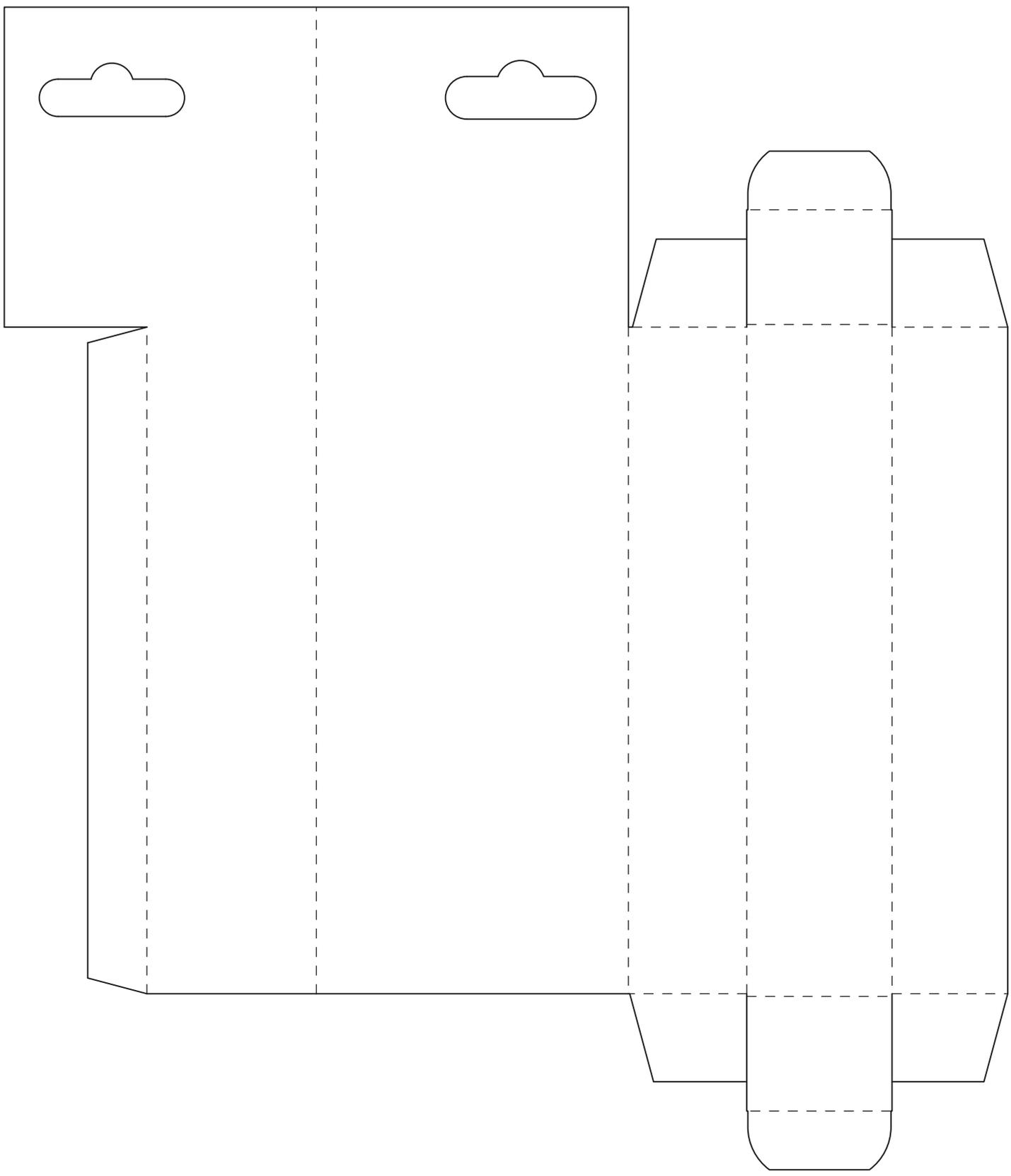

EN General advice: Carefully read the instructions for use prior to using naildoc® Rapid Fungal Nail Treatment Pen, even if you have used this medical device before. The information may have been amended according to new research and findings in the time since you opened your last package. This medical device can be obtained without a prescription. Nevertheless, naildoc® Rapid Fungal Nail Treatment Pen should be used with great care in order to get a good result. Keep these instructions for use in case you do not consult them again.

Description and intended usage: naildoc® Rapid Fungal Nail Treatment Pen is a medical device, which has been clinically proven to be effective in the treatment of fungal nail infection (Onychomycosis). The unique formula works immediately and kills fungi on the nails surface within 20 seconds. It penetrates through the nail deep into the nail plate. Thanks to this instant effect the contamination of other nails could be prevented. It also protects against recurrence, as the brush will not be infected with the fungi. The liquid forms a layer with fungistatic properties which creates an unfavourable environment for the development of fungi, and thus creating a physical barrier against spread of fungi.

As one of the causes of fungal nail infections is a moist environment, it is highly recommended to always wear socks of absorbent and natural material such as cotton or wool. Socks made of these natural materials absorb more moisture. This is very important as fungi like high moisture environments so to use them can multiply. Wearing of fungal nail, it is important to keep feet dry.

Mode of action: Thanks to a combination of ingredients, naildoc® Rapid Fungal Nail Treatment Pen forms a layer that creates an unfavourable environment for the development of fungi in the plate of the nail. It functions as a physical barrier that prevents the spreading of fungi. Urea allows the active ingredients to penetrate deep into the nail plate, naildoc® Rapid Fungal Nail Treatment Pen reduces the acidity of the nail, creating an unfavourable environment, in which the fungi can no longer grow and spread. In addition, studies show that tea tree oil inhibits the development of hyphal threads.

Dosage and use: naildoc® Rapid Fungal Nail Treatment Pen is intended for adults and external use only.
• Carefully twist the back of the pen until a drop of liquid is released.
• Apply the product two or three times a day to the affected nail until the new nail has completely grown out.
• Nail polish must be removed before the naildoc® Rapid Fungal Nail Treatment Pen can be applied.
• naildoc® Rapid Fungal Nail Treatment Pen can be combined with oral antimycotics.

Nail fungus is stubborn to remove, so it is very important to keep using naildoc® Rapid Fungal Nail Treatment Pen until full regrowth of the nail. To achieve good results the best treatment method is to use it for 3 months.

Ingredients naildoc® Rapid Fungal Nail Treatment Pen: Aqua, Ethoxydiglycol, Urea, Pentylene Glycol, Alcohol Denat., Tridiceth-9, PEG-40 Hydrogenated Castor Oil, Benzyl Alcohol, Melaleuca Alternifolia Leaf Oil, Limonene, Glycerin, Nalilact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrid Oil, Lactic Acid, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cabin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

Content: 4 ml

Do not use naildoc® Rapid Fungal Nail Treatment Pen in the event of:

- oversensitivity for one or more of the ingredients;
- if you doubt whether you can use and/or can use it in combination with other medication, then please consult with your doctor or pharmacist.

Use during pregnancy and/or breastfeeding: No information is available regarding the use of naildoc® Rapid Fungal Nail Pen during pregnancy or the breastfeeding phase. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using.

Side effects:

- Possible side effects include: erythema, irritation, edema, pain, nail discoloration and onycholysis, allergic reactions.

Warnings:

- For external use only.
- Check the list of ingredients to know if you are allergic to any of the ingredients contained in the gel.
- Avoid contact with eyes or mucous membranes.
- Do not apply to open wounds or damaged skin. Do not bandage tightly.
- Do not apply heat to the area of application.
- If you experience allergic reactions, stop using the product.
- Keep out of reach of children. Not suitable for babies and toddlers.
- In case of no effect, consult a physician.

Storage method: Store at room temperature (5-25°C) in a dry and dark place in the original container and out of reach of children.

For the expiry date please see the bottom of the packaging.

Medical device: naildoc® Rapid Fungal Nail Treatment Pen is a class IIA medical device according to the definition of the Council Directive 93/42/EEC.

PK Benelux BV
Vluchtoord 17,
5406 XP Uden, NL

Produced for: doc phytolabor
guterat Gesundheitsprodukte GmbH & Co. KG
A-6020 Innsbruck, Eduard-Bodem-Gasse 6, Austria

Date on which these instructions for use were last revised: March 2020
-091648v1.0 -REV01/ 3-2020

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Sestavine: Aqua, Ethoxydiglycol, Urea, Pentylene Glycol, Alcohol Denat., Tridicheth-9, PEG-40 Hydrogenated Castor Oil, Benzyl Alcohol, Melaleuca Alternifolia Leaf Oil, Limonene, Glycerin, Naltilact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrid Oil, Lactic Acid, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cablin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarnin, Citric Acid, Farnesol.

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guterrat Gesundheitsprodukte GmbH & Co. KG