Sonoma  
PHARMACEUTICALS  
A Relentless Passion For Healing

# Microdacy<sub>n</sub><sup>60</sup>® Wound Care

Oculus Technologies of México S.A. de C.V.  
Industria Víidriera No. 81, Industrial Zapopan Norte  
Zip Code 45130, Zapopan, Jalisco, México.  
Phone: +52 33 38 33 67 22

EC REP  
Sonoma Pharmaceuticals Netherlands B.V.  
(Formerly Oculus Innovative Sciences Netherlands B.V.)  
Boven de Wolfskull 3, C30-C32  
6049 XH Lerten/Roermond  
The Netherlands  
Phone: +31 (0)475 318 666  
Fax: +31 (0)475 318 342  
www.sonomapharma.eu

I00R-003 Rev 09  
Jul 16<sup>th</sup> 2018

## Microdacy<sub>n</sub><sup>60</sup>® Wound Care

Instructios for use

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Bruksanvisning

Bruksanvisning

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Navodila za uporabo

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Kasutusjuhend

## Instructions for use:

**Indications for use:** Microdacy<sub>n</sub><sup>60</sup>® Wound Care is an electrolysed solution intended for use in the debridement and moistening of acute and chronic wounds, ulcers, cuts, abrasions and burns including those located in any human cavity such as the oral, nasal or ear. Sodium Hypochlorite and Hypochlorous acid are ancillary substances may have a local antimicrobial effect. Through reducing the microbial load and assisting in creating a moist environment, it enables body to perform its own healing process. Microdacy<sub>n</sub><sup>60</sup>® Wound Care can be broadly applied within a comprehensive wound treatment and is safe on mucous membranes.

**Handling:** Microdacy<sub>n</sub><sup>60</sup>® Wound Care is biocompatible for its indicated use, non-irritating to human skin, eyes or throat, and is not a skin sensitizer. No special handling precautions are required.

**Storage:** Microdacy<sub>n</sub><sup>60</sup>® Wound Care should be stored in its original sealed container at ambient conditions. Store at room temperature and prevent product from freezing. It is non-flammable. After use, the container should be closed again, before storing.

**Stability:** Until opened, the stability of Microdacy<sub>n</sub><sup>60</sup>® Wound Care is assured up to expiration date (see bottle for year and month of expiration "YYYY MM"). Upon opening, note the date on the bottle in the section "Date opened". Stability of the unused solution is assured for up to 60 days after opening.

**Directions for use:** Microdacy<sub>n</sub><sup>60</sup>® Wound Care is ready for use with no mixing or dilution required. Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Apply Microdacy<sub>n</sub><sup>60</sup>® Wound Care directly to the affected area. Use Microdacy<sub>n</sub><sup>60</sup>® Wound Care at each wound dressing change. Saturation of the dressing with Microdacy<sub>n</sub><sup>60</sup>® Wound Care will assist with dressing removal.

**Treatment:** Prior to cleanse the area or use the product, please carried out an appropriate handwashing. Soak/saturate dressing with Microdacy<sub>n</sub><sup>60</sup>® Wound Care at each dressing change, as and when required. Immersion is allowed for maximum 15 minutes, daily, during initial treatment.

### Warnings and points to note:

- If tamper resistant seal is broken or missing, do not use.
- For external use only, not for injection or consumption.
- Do not use if hypersensitive to product ingredients.
- Do not use Microdacy<sub>n</sub><sup>60</sup>® Wound Care in combination with other topical solutions without the advice of a healthcare professional.
- In case of deep or serious wounds, or if you have any other treatment concerns consult a healthcare professional.
- If signs of infection occur (increased pain, bleeding, pus) or if the wound does not show signs of healing, consult a healthcare professional.
- Report any adverse reactions to a healthcare professional.
- In addition to using Microdacy<sub>n</sub><sup>60</sup>® Wound Care systemic and/or local antibiotics may be required in some cases.
- Keep out of the reach and out of sight of children.
- Keep container closed when not using.
- Microdacy<sub>n</sub><sup>60</sup>® Wound Care is harmless even in case of accidental contact with eyes, nose or ingestion.

**Disposal:** The solution is non-toxic and safe to the environment. No special disposal considerations required.

**Ingredients:** Electrolysed Water, Sodium Chloride, ancillary substances: Sodium Hypochlorite and Hypochlorous Acid.

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