SLO

vivamel
medicinski med

GB

vivamel
medical honey

D

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Navodilo za uporabo VIVAMEL – MEDICINSKI MED 20 g

Pred prvo uporabo izdelka Vivamel medicinski med 20 g natančno preberite navodilo. Navodilo shranite. Morda ga boste želeli ponovno prebrati.

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CE 1304

Pomen simbolov:

Instructions for use VIVAMEL – MEDICAL HONEY 20 g

Carefully read the instructions before using the Vivamel medical honey 20 g for the first time. Keep the instructions. You may want to read them again in the future.

WHEN TO USE THE VIVAMEL MEDICAL HONEY 20 g?

Vivamel medical honey is intended for care and healing of wounds, acute and chronic, as well as surface wounds, abrasions and cuts, burns, diabetic ulcers, infected and deep wounds.

HOW DOES THE VIVAMEL MEDICAL HONEY 20 g WORK?

Vivamel chestnut medical honey in tube has a high sugar content (80 % minimum), low water content (17 % maximum) and acidic pH. Due to physical properties it has a high osmotic activity, which creates an unfavourable environment for the growth of microorganisms, and also facilitates wound healing. Honey provides moist healing and forms a mechanical protection on the wound.

HOW TO USE THE VIVAMEL MEDICAL HONEY 20 g?

1. First clean the wound under running tepid tap water or with a physiological saline.
2. Gently dry the wound and the area around it.
3. When you open the tube for the first time, pierce the seal of the tube by inverting the cap, and press a small quantity of honey on a sterile compress and discard it. Apply the medical honey from the tube to the pad of the sterile bandage or to a sterile primary dressing of your choice; for example, alginate dressing, non-woven compress, polyurethane foam or any other appropriate dressing.
4. Place the pad of the bandage or the dressing so as to fully cover the wound. Fix the adhesive part of the bandage on the surrounding area of the skin. Secure the non-adhesive dressing with an appropriate retention bandage. In case of excessive wound drainage, place a highly absorbent dressing over the primary dressing and secure it with a retention bandage.

Initially use the dressing once a day, and later every two to three days, depending on the quantity of wound drainage. Follow the procedure described above.

WHEN NOT TO USE THE VIVAMEL MEDICAL HONEY 20 g?

If you are allergic to honey, do not use the Vivamel medical honey.

WARNINGS

Always wash your hands before and after using the product. The Vivamel medical honey 20 g is not intended for ingestion. If ingested, it may cause blood sugar in diabetics to rise or hypersensitivity reaction in people allergic to honey. Avoid contact with eyes or mucosa. If the product comes into contact with eyes or mucosa, rinse with abundant quantities of water. Use up the Vivamel medical honey 20 g within four months after first opening the tube. After every use close the tube well in order to prevent honey from leaking out and to prevent possible ingress of air and microorganisms into the tube. The product is sterile only until it is opened for the first time. If the tube is damaged, don't use it. Honey from the tube may be applied only on a sterile pad or a sterile primary dressing in order to prevent contamination of the remaining tube content.

Clinical trials of wound healing with Vivamel medical honey in diabetics have shown very good results, but treatment should certainly take place under doctor's supervision.

WHAT ARE POSSIBLE SIDE EFFECTS?

Due to high osmotic activity of honey, which is based on removal of water molecules from the wound, a burning sensation or a mild pain may appear after applying honey on the wound, which usually goes away after 30 minutes to two hours maximum after the application. If the pain is unbearable, stop using the Vivamel medical honey 20 g. Notify your doctor or pharmacist about any other unwanted side effects.

HOW TO STORE THE VIVAMEL MEDICAL HONEY 20 g?

Store the Vivamel medical honey 20 g in a dry place at a temperature of 5° C to 35° C in a horizontal position or with the cap upwards. Do not place the tube on the cap, because next time you use the honey it will leak out uncontrollably. Do not expose the product to direct sunlight.

VIVAMEL – MEDICAL HONEY 20 g
Vivamel – Medical honey 20 g; Art. No.: 1052843

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