Test za samotestiranje na celijakiju

Gluten je proteinski kompleks prisutan u pšenici, ječmu, raži i u mnogim drugim žitaricama, kao i u hrani koja sadrži žitarice. Intolerancija na gluten, ili celijačna bolest (celijakija), je težak gastrointestinalni poremećaj, koji traje doživotno, i koji izaziva mnoštvo simptoma kao što su dijareja, nadutost stomaka, gubitak težine, neuhranjenost, oboljenja kože, a ispoljava se kroz imuni odgovor u kome se proizvode specifična antitela (IgA i IgG). Tokom bezglutenske ishrane dolazi do smanjenja simptoma i nivoa IgA i IgG antitela povezanih sa celijakijom. Često se nivo ovih antitela više ne može detektovati već nakon nekoliko nedelja – a najkasnije do 6 nedelja od uvođenja novog načina ishrane. Stoga ako sledite bezglutensku ishranu moguće je da ćete da dobijete negativan rezultat. Primenu bezglutenske ishrane treba da sprovedite samo prema savetu lekara.

Xeliac® Test Pro je jednostavan, brz i pouzdan test za samotestiranje, koji detektuje prisustvo intolerancije na gluten uslovljene IgA i IgG anti-transglutaminaznim antitelima u uzorku krvi iz prsta.

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Gluten is a protein complex present in wheat, barley, rye, in many other cereals as well as in all the food containing cereals. Gluten intolerance, or coeliac disease, is a severe gastrointestinal disorder persisting all life and causing a variety of symptoms like diarrhoea, abdominal distension, weight loss, malnutrition, skin diseases and manifests with an immune response with the production of specific antibodies (IgA and IgG). During a gluten-free diet, both symptoms and IgA/IgG antibodies levels associated to the coeliac disease, decrease. Quite often these antibodies are not detectable anymore, already after a few weeks, at least within 6 weeks, under the new dietary regimen. Therefore, if complying to a gluten-free diet, the test may result negative. Complying to a gluten-free diet must be done only after medical advice. **Xeliac® Test Pro** is a simple, rapid and reliable self test detecting the presence of gluten intolerance-associated IgA and IgG anti-transglutaminase antibodies for a fingertip drop of human blood. Should the test be positive, a doctor must be consulted in order to undergo further analysis that will either confirm or exclude the coeliac disease.

TEST PROCEDURE

Carefully read the test instructions and precautions before use. For the execution of **Xeliac® Test Pro**, a watch will be needed. The test requires only a drop of blood and the result will be available in 3-5 minutes. The test shall display a result within 10 minutes.

TEST COMPONENTS

Aluminium pouch containing test cassette with dessicant

2 sterile single use lancets for blood sampling
CE 0197 (MD) SteriLance Medical (Suzhou) Inc., No.168 PuTuShan Road, New District, 215153, Suzhou, Jiangsu, P.R.China.

2 micropipettes for blood collection
CE (IVD) SAFE-TEC® Clinical Products LLC, 500 Parkway, Broomall, PA 19008, USA

Single use dropper bottle with test diluent

Plaster
CE (MD) ZCC GmbH Ammonitenweg 16, D-89081 Ulm, Germany

STORAGE






All the test components must be stored at room temperature (+8° + 25° C). The test must be used before the date printed on the box.

PRECAUTIONS

- Second lancet and second micropipette shall be used only if/when the first fails.
- If the instructions for use are not followed carefully, test may give false results.
- A final diagnosis has to be performed by a physician on the basis of clinical symptoms and with support of further laboratory analysis.
- A doctor shall be consulted before starting a gluten-free diet.
- The test shall not be used after the expiry date printed on the box.
- The test shall not be used if the aluminium pouch is damaged.
- All the test components shall be handled with care. Test diluent (DIL) contains Proclin 300. May produce an allergic reaction. Do not ingest.
- Do not reuse. All the components are single use.
- **Xeliac® Test Pro** has a 96.5% sensitivity and a 97.3% specificity.
- Alcohol or local anaesthetics do not alter the test.
- The test may result negative only in limited cases, for instance with low antibody titre.
- After use, all the test components should be placed inside the test box and disposed of with the normal household waste.
- Keep **Xeliac® Test Pro** out of reach of children.
- Contains desiccant. Do not ingest.

TEST PROCEDURE

Before performing the test, ensure all the test components are accessible.

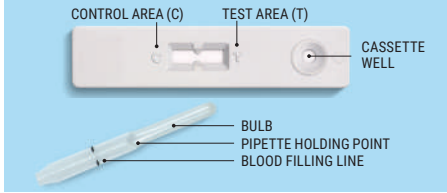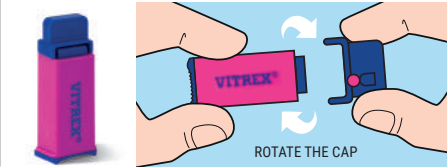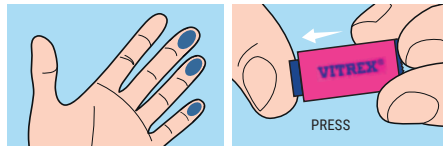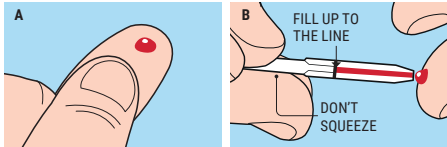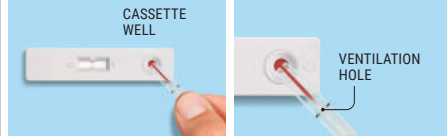- 1 Extract the test cassette from the aluminium pouch and lay it on a flat and dry surface. Prepare the pipette and place it in an easy accessible position. **The test must be performed within 10 minutes.**

- 2 A. Take the lancet as indicated in the picture.
 B. Twist the blue safety cap and extract it completely. The lancet is now ready for use. **Do not use the lancet (sterile) if the cap has been removed or if it is missing.**

- 3 Wash your hands with warm water and shake them downwards to let blood flow. Press the part of lancet devoid of the safety blue cap firmly against the clean finger (in one of the areas indicated in the picture). The puncture is painless. **Warning: Mildly press the fingertip before using the lancet.**

- 4 A. Immediately squeeze the fingertip until a conspicuous drop of blood appears.
 B. Touch the end of the pipette against the blood drop. The blood will be drawn automatically into the pipette (filling up by capillarity) **until reaching the line on the bulb. Do not squeeze the bulb during this step.**

- 5 Once filled up, completely empty the pipette into the round well on the cassette labelled by an arrow by squeezing the bulb at the end of the pipette. Ensure all the blood in the pipette is removed and if necessary also squeeze the tube of the pipette and cover the ventilation hole on the nozzle with a finger.
 

⌚ WAIT 60 seconds.
 In the meantime, apply the plaster on the fingertip.
- 6 Open the test diluent bottle (DIL) as indicated by the arrows and dispense **2 drops** into the same well used for the blood. **Warning: Do not dispense more than 2 drops of diluent, otherwise the interpretation of the test may be compromised.**

⌚ After this step **WAIT from 3 to 5 minutes** and then read the result. The result must appear within 10 minutes. Do not read the result after 10 minutes, the test wouldn't be reliable.

READING THE RESULTS

- Negative Result:**
 Only a blue coloured line appears near to the 'C' control mark on the cassette. This is the control blue line and must always appear. In this case the presence of antibodies associated to the coeliac disease can be excluded. In the presence of gastrointestinal symptoms, perform further medical investigations.
- Positive Result:**
 Close to the control blue line also a red line of variable intensity appears near the 'T' test mark. This indicates the presence of antibodies associated with the coeliac disease. Local doctor shall be contacted for further laboratory investigations.
- Non Valid Result:**
 If there is no control blue line, the test is to be considered invalid, even if a red line appears. The non valid result could be due to a non correct procedure, a deterioration of the reagents or to a non correct amount of blood analysed. The test must be repeated.